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WARRANTY

The product has a 24-month warranty, valid from the date it is delivered by CA.MI. snc. During the guarantee period free repair and / or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc. Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by CA.MI, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.

The guarantee will be invalidated if the registration number has been removed, cancelled or altered.

CA.MI. snc cannot be held liable for accidental or indirect damages should the device be modified, repaired without authorization or should any of its component be damaged due to accident or misuse.

This device is recommended for the prevention of basic bedsores in long-term and home therapy. Skin is delicately massaged at alternating points of contact for improved blood flow and tissue oxygenation to prevent the formation of bed sore ulcers. It is easy to install, use and maintain and offers pressure adjustment based on patient weight. This non-toxic PVC mattress is inflated with its own compressor and supports patients lying down. Check the correct functioning of the regulator by turning the knob from MIN to MAX.

IMPORTANT SAFETY RULES

1. Particular precautions must be made concerning electromagnetic compatibility. The medical device must be installed and used according to information supplied with the accompanying documents;
2. On opening the packaging, check the integrity of the appliance, paying particular attention to the presence of damage to the plastic parts, which may make access possible to internal live parts and also to breakage and / or peeling of the power supply cable. **In these cases don't connect the plug to the electric socket. Carry out these controls before each use;**
3. Before connecting the appliance always check that the electric data indicated on the data label and the type of plug used, correspond to those of the mains electricity to which it's to be connected;
4. If the plug supplied with the appliance is incompatible with the mains electricity socket, contact qualified staff for replacement of the plug with a suitable type. The use of simple or multiple and / or extension adapters is not generally recommended. Whenever their use is indispensable, use those in compliance with safety regulations, however paying attention not to exceed the maximum power supply limits, which are indicated on the adapters and extensions;
5. Respect the safety regulations indicated for electrical appliances and particularly:
 - Use original components and accessories provided by the manufacturer CA.MI. to guarantee the highest efficiency and safety of the device;
 - Never immerse the appliance into water;
 - Preserve and use the medical device in environments protected from atmospheric factors and at a distance from heat sources;
 - Keep off the reach of children or not capable people without supervision;
 - Don't leave the appliance connected to the power supply socket when not in use;
 - Don't pull the power supply cable to disconnect the plug remove the plug from the mains socket correctly;
 - Preserve and use the medical device in environments protected from atmospheric factors and at a distance from heat sources;
 - In case of failure and / or malfunction please contact immediately the technical service CA.MI. Failure to comply with the above can jeopardise the safety of the device; None of electric or mechanical parts have been designed to be repaired by customers or end-users. Don't open the device, do not mishandle the electric / mechanical parts. Always contact CA.MI. technical assistance
6. **This medical device must be destined exclusively for the use for which it has been designed and described in this manual.** Any different use must be considered incorrect and therefore dangerous; the manufacturer cannot be considered liable for damage caused by improper, incorrect and / or unreasonable use or if the appliance is used in electrical plants that are not in compliance with the regulations in force;
7. Instrument and accessories discharging must be done following current law regulations in every country of use.
8. Using the device in environmental conditions different than those indicated in this manual may harm seriously the safety and the technical characteristics of the same.

READ INSTRUCTION MANUAL CAREFULLY BEFORE USE

**THE INSTRUMENT MUST NOT BE DISASSEMBLED.
FOR A TECHNICAL SERVICE ALWAYS CONTACT CA.MI.**

KEEP OFF REACH OF CHILDREN OR NOT CAPABLE PEOPLE WITHOUT SUPERVISION

IMPORTANT INFORMATION FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EC DIRECTIVE 2002/96/EC:

In respect of art. 13 Decreto Legislativo 25 Luglio 2005, n.151 "Actuation of European directives 2002/95/EC, 2002/96/EC and 2003/108/EC, for reduction in use of dangerous substances in the electric and electronic device and for garbage disposal". The symbol as over applied on the device or its packaging means that at the end of its useful life the product must not be disposed of with domestic waste. At the end of device useful, the user will must deliver it to the able collecting centres for electric and electronic garbage, or give back to the retailer in the moment of equivalent new device purchasing, one against one. Disposing of the product separately prevents possible negative consequences for the environment and for health, deriving from inadequate disposal. It also allows the recovery of materials of witch it's made up in order to obtain an important saving of energy and resources and to avoid negative effects to the ambient and health. In case of abusive disposal of device by user, will be applied administrative endorsements in compliance with current standard.

TECHNICAL CHARACTERISTICS

Typology (MDD 93/42/EEC)	Medical Device Class I
PUMP	MATTRESS
Power supply: 230V~ - 50Hz	Size: 3000(L)mm x 900 (W)mm x 65(H)mm
Air output: 4.5 liter/min.	130 bubble cells
Pressure range: 30 mmHg ÷ 110 mmHg	Material: PVC (EN-71, low toxic)
Cycle time: 10 min.	PVC thickness: 0.30 mm
Power consumption: 5W	Color: beige
Fuse: 1A	Repair kit included
Size: 240(L)mm x 120 (W)mm x 95(H)mm	Without vents and with end flaps
Weight: 1.4 Kg	
Working Condition	Room temperature: 10°C ÷ 35°C (50°F ÷ 90°F) Humidity: 20 ÷ 80% RH
Conservation condition and transport	Room temperature: -15°C ÷ 50°C (5°F ÷ 122°F) Humidity: 10 ÷ 90% RH

MADE IN CHINA

SYMBOLOLOGY

	CE marking in conformity with EC directive 93/42/EEC Manufacturer: CA.MI. snc di Attolini Mario & C. – Via Ugo La Malfa 31 – 43010 Pilastro (PR) Italia
	Warning, consult the instruction manual
	To preserve in place coolness and dry land
	Conservation temperature
	Type BF equipment
	Fuse
~	Alternate Current
Hz	Mains Frequency
ON	ON
OFF	OFF

STANDARD FEATURES

- ◆ Simple to operate.
- ◆ Portable and lightweight.
- ◆ Pad consists of 130 bubble cells.

CLEANING AND MAINTENANCE

The medical device does not need maintenance or lubrication.

Before use always check correct functioning and safety of the device.

Unpack the instrument and always check integrity of plastic parts and feeding cable, they might have been damaged during previous use. Connect cable to electrical network and turn switch on.

Before using the device on people carefully clean the mattress with neutral detergent.

Never immerse the compressor into water. Drain the mattress after each wash .

MATTRESS: A first cleaning of the mattress can be made using a damp cloth.

Do not use abrasive cloths and / or cutting surfaces that might damage the surface of the mattress.

COMPRESSOR: Cleaning the compressor with damp cloth and very clean.

Never immerse the compressor into water. Check periodically that the connectors are not clogged by dust.

WARNING: All the cleaning and maintenance operation must be made after disconnected the compressor from the electrical network.

**DO NOT IMMERSE THE PUMP IN WATER
DO NOT HEAT OR STEAM IN AUTOCLAVE**

TROUBLESHOOTING:

PROBLEM	SOLUTION
<i>The compressor will not turn.</i>	<i>Verify the plug may be misplaced in the wall socket Check that you have not missed a fuse.</i>
<i>The mattress doesn't take the form</i>	<i>Check that the basis of the mattress is firmly attached to the items. Make sure the mattress is firmly attached to the bed.</i>

OPERATING INSTRUCTIONS

1. Position pump on horizontal surface or hang it by means of the hook.
2. Open pad on the bed. Secure it with end flaps or straps. Air tubes must be at foot end of bed.
3. Connect air feed tubes to air outlet on the pad and pump, ensuring that the tubes are not "kinked" or twisted. Allow space for bed rails to drop freely.
4. Plug the pump into a suitable electrical power source and switch on. The switch indicator should now illuminating.
5. Allow 20 minutes for pad to inflate fully.
6. Cover with a sheet and tuck loosely.
7. Please use pressure adjustable knob to give maximum patient comfort.